


Dominant 35c/i

For gastrointestinal suction procedures:

Check the patient's condition and the proper running of the pump **at least every four hours** by means of the following visual signs:

Warning note:

For use only by persons properly trained in suction techniques and the operation of suction equipment. Improper use or selection of suction mode may cause serious injury.

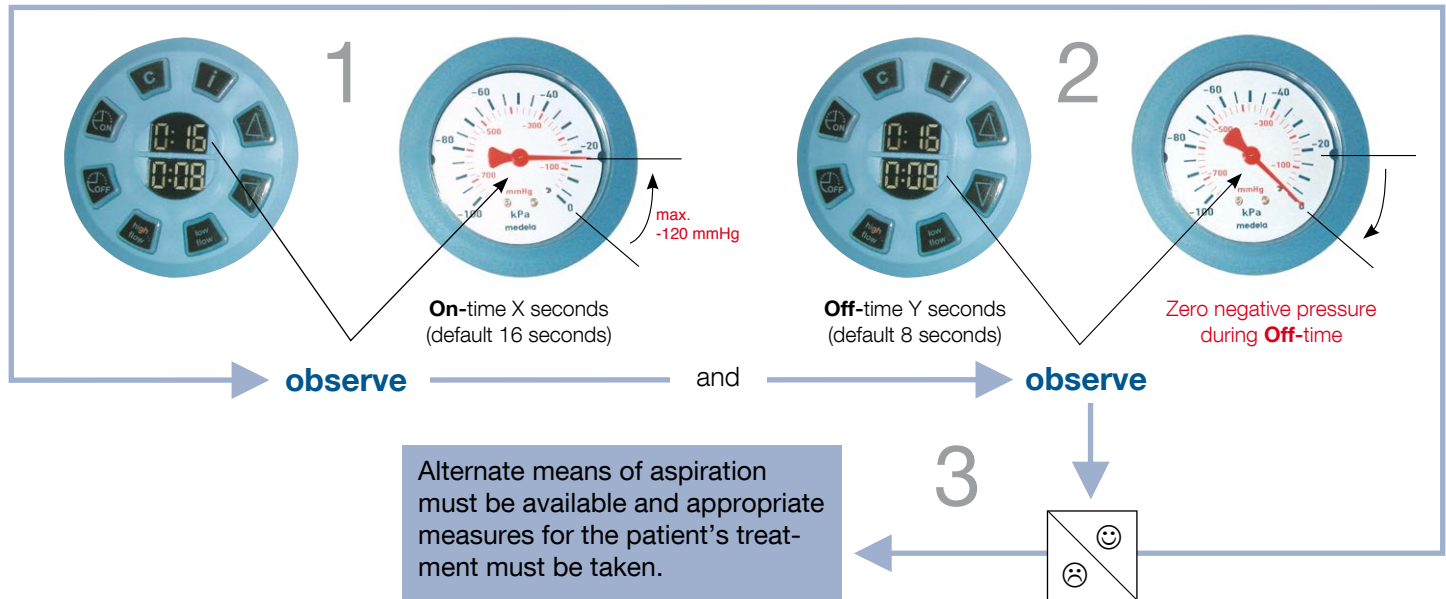


Dominant 35c/i

How to use the Control Panel for applications in the intermittent suction mode:

Application

intermittent

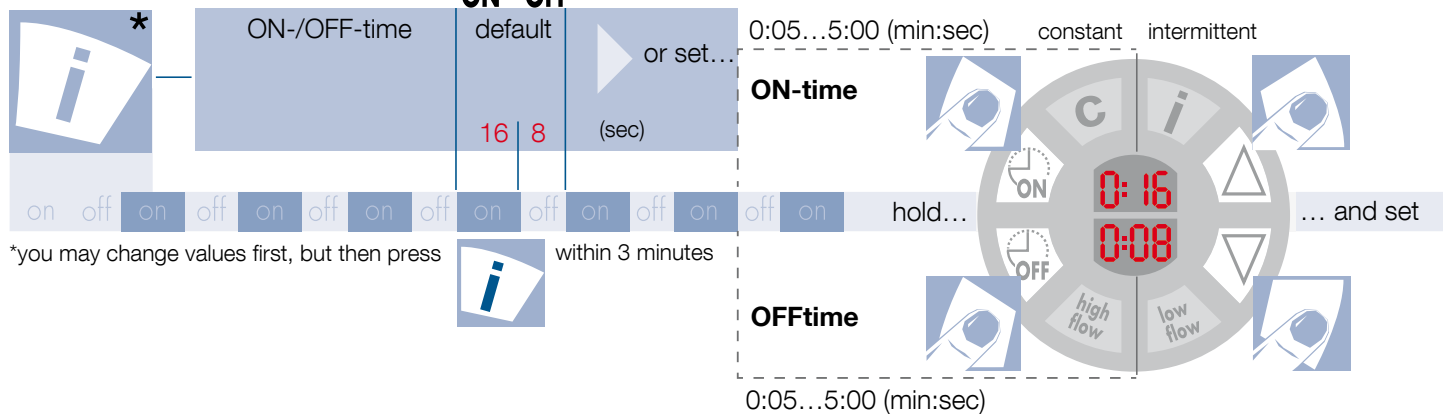

switch **ON** by selection mode



CAUTION: Activating the pump via Footswitch Causes Operation in Constant Mode – High Flow



*you may change values first, but then press

within 3 minutes

This Quick Card in no way replaces the instructions for use REF 200.0682



or



or



switch **OFF**

Medela AG, Medical Technology, Lättichstrasse 4b, 6341 Baar / Switzerland
Phone +41 (0)41 769 51 51, Fax +41 (0)41 769 51 00, ism@medela.ch, www.medela.com

medela